


Two types of armrest are available: Standard armrest (A) and Wide armrest (B).

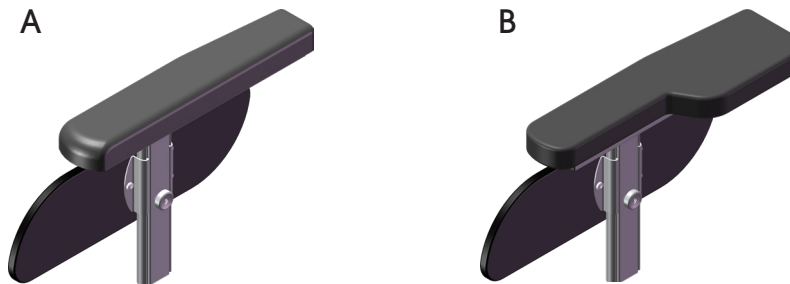

Standard armrest


The standard armrest consists of an arm support and a side support. The height is adjustable (10 settings) and the depth is adjustable (7 settings). Both height and depth are adjusted without tools. A graded scale is included for the height setting.

Wide armrest

The wide armrest consists of a padded arm support and a side support. The armrest pad is wider at the back and is softer than the standard armrest. The wide armrest is adjusted in the same way as the standard.


Side support padding

Side support padding (C) is a thin cover that fits over the side support of the armrest and is closed with a zipper. It is used for users who require a softer surface against the leg.

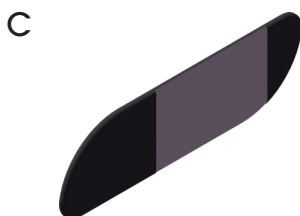


Measurements - Armrests


Standard arm support

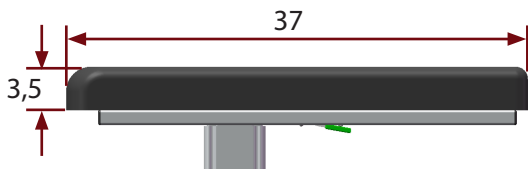


Wide arm support

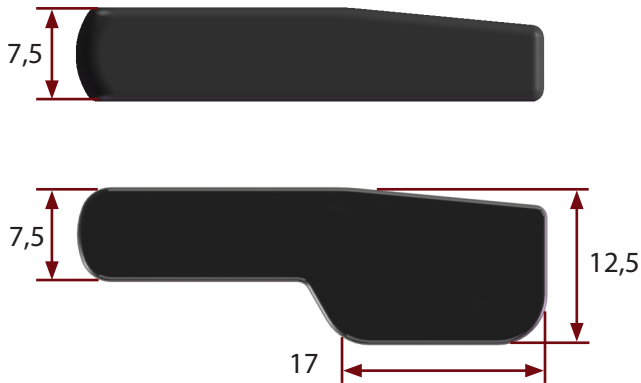

Rear position

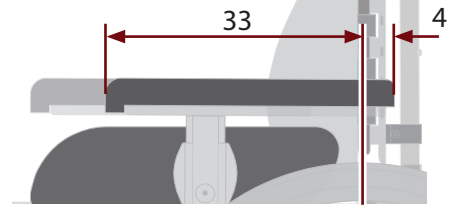


Front position

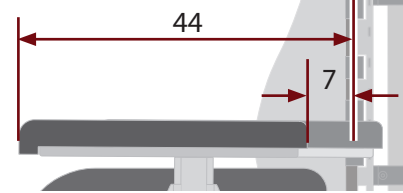


Height - Armrest

Table 1 - Armrest height a [cm]

Model	Lowest	Highest
Standard	25,5	34,5

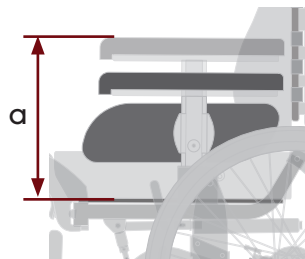

Measurements - Side Support

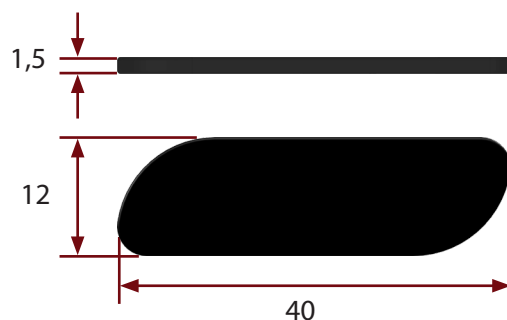

Keep in mind

- The overall width of the wheelchair is not increased with use of the Wide armrests.
- The depth adjustment of the tray table may be somewhat reduced if the wheelchair is equipped with Wide armrests.

Article numbers

All article numbers are found in the Prescription Form, doc.nr. 97750-1.