

Seat cushion (A) for HD Force comes in two versions, Firm and Soft. The seat cushion is normally placed centred on the seat, so that the seat plate is completely covered. The cushion is attached to the seat with velcro.



### Seat cushion

The seat cushion is constructed with several layers of foam, each with a different firmness. The nominal thickness of the cushion is 7 cm, with a width that suits the model of wheelchair. The choice of Firm or Soft cushion version should be made according to the needs of the user and with pressure measurements.

### Covers

Seat cushions are fully covered in stretchy black velour. The pile of the fabric on the seating surface is aligned to hinder sliding forward and to facilitate moving sideways and back.

The cover is removable and is washable at 60° C. Covers can also be ordered separately.

### Incontinence protection

A cover in the form of a cap is available to protect against incontinence. The cover is washable at 60° C.

### Keep in mind

- Seat cushions have a limited lifespan depending on use. Cushions should be checked regularly for the comfort and safety of the user. Cushions should be replaced when they become worn and flattened.

### Article Numbers

All article numbers are included in the Prescription Form, document number 97750-1.