

Bag hook

Doc. nr. 97912-1 Rev A 2024-11-26

Page 1 (1)

Bag hooks (A) attach to the vertical portions of the push bar and are used to hang lighter bags or other items. To not interfere with the movement of the wheels, bags should be placed in the middle of the back support with a handle on each bag hook.

A



Keep in mind

- Tipping risk is increased if the wheelchair seat and/or back are tilted back and the bag hooks are loaded.
- Bag hooks should not be loaded with more than 5 kg total.
- Check that the wheelchair is equipped with tip protectors and that they are in the active positions.
- Responsibility lies with the caregiver and user to check tipping risk and ensure tip protectors are in place.

Article Numbers

All article numbers are included in the Prescription Form, document number 97750-1.