


## IV Pole

HD Force can be equipped with an IV pole. This can make it easier for the user to be more mobile for some types of treatments and examinations.

The construction is sturdy and consists of a lightweight aluminium pole (A) and a mount (B) which is attached to the back frame of the wheelchair on the right or left side. The length of the IV pole is easily adjusted with a knob. The angle is controlled using a knob on the mount, so that the pole can always be held vertical even if the wheelchair seat is tilted and/or the back reclined. The pole has two hooks as standard to be able to hold two drip bags.

The IV pole mount is labelled with a warning symbol to indicate that important information should be noted. See *Risk Information*, document number 97711-1.

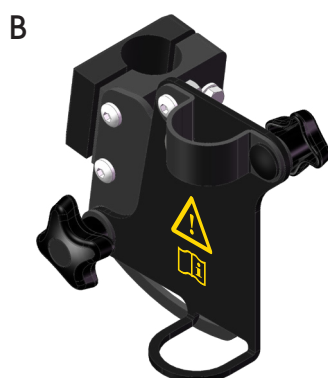

### Keep in mind

- Always keep the IV pole as vertical as possible.
- Always check that the knob is tightened before use, otherwise the IV pole can fall forwards or backwards.
- Using an IV pole increases tipping risk, especially with heavy drip bags and with the seat or back tilted.
- Be aware of doorway heights when an IV pole is used.
- Always use tip protectors.

### Article Numbers

All article numbers are found in the Prescription Form, document number 97750-1.